

Chlamydia and Neisseria Gonorrhoea – Non-Urogenital

| | |
|--------------------------------|--|
| Synonyms | STD, STI CTNG |
| Abbreviation | CHLAMAGPCR |
| Source: | Rectal or Throat |
| Collection requirements | <p><u>Rectal:</u></p> <ol style="list-style-type: none">1. Rectal Swab collected approx. 1cm beyond the anal sphincter stored in Xpert Swab Transport Reagent tubes<ol style="list-style-type: none">a. Stable up to 60 days at 2°C - 30°C.b. Highly soiled fecal swabs should not be used as they may result in errors. <p><u>Throat:</u></p> <ol style="list-style-type: none">1. Throat Swab collected from the pharynx including tonsils, posterior wall, and uvula stored in Xpert Swab Transport Reagent tubes <p>Stable up to 60 days at 2°C - 30°</p> |
| Reference range | Negative |
| Routine / STAT T.A.T | Within 24 hours / 4 hour |
| LAB code | 13387 |